

Properties

Prio Recon	Off
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Wait for user to start	Off
Start measurements	single

Routine

Nr. of slice groups	1
Slices	35
Dist. factor	25 %
Position	L1.2 P15.7 H3.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	200 mm
FoV phase	100.0 %
Slice thickness	1.5 mm
TR	4500.0 ms
TE	108.0 ms
Averages	1
Concatenations	2
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast

TD	0.0 ms
MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	None
Water suppr.	None
Restore magn.	Off
Averaging mode	Long term
Measurements	1
Reconstruction	Magnitude
Multiple series	Each measurement

Resolution

Base resolution	512
Phase resolution	100 %
Phase partial Fourier	Off
Trajectory	Cartesian
Interpolation	Off
PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	27
Reference scan mode	Integrated
Image Filter	Off
Distortion Corr.	Off
TD	0.0 ms
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Nr. of slice groups	1
Slices	35
Dist. factor	25 %
Position	L1.2 P15.7 H3.0 mm
Phase enc. dir.	R >> L
Phase oversampling	0 %
Multi-slice mode	Interleaved
Series	Interleaved
Nr. of sat. regions	0
Position mode	L-P-H
Fat suppr.	None
Water suppr.	None
Special sat.	None
Special sat.	None
Table position	P
Restore magn.	Off

System

Body	Off
HEP	On
HEA	On
Position mode	L-P-H
Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect
B0 Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto
? Ref. amplitude 1H	0.000 V
Position	Isocenter
Rotation	0.00 deg
R >> L	350 mm
A >> P	263 mm
F >> H	350 mm
Frequency 1H	123.236072 MHz
Correction factor	1
Excit 1H	272.908 V
Gain	High
Table position	0 mm
Img. Scale. Cor.	1.000

Physio

1st Signal/Mode	None
Magn. preparation	None
Dark blood	Off
Trajectory	Cartesian
Resp. control	Off

Inline

Distortion correction	Off
-----------------------	-----

Sequence

Introduction	Off
Dimension	2D
Compensate T2 decay	Off
Averaging mode	Long term
Multi-slice mode	Interleaved
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	222 Hz/Px
Flow comp.	No
Allowed delay	60 s
Echo spacing	12 ms
Define	Turbo factor
Turbo factor	18
Echo trains per slice	15
RF pulse type	Normal
Gradient mode	Normal
Hyperecho	Off
TX/RX delta frequency	0 Hz
TX Nucleus	None
TX delta frequency	0 Hz
Coil elements	HEA;HEP
Acquisition duration	0 ms
Mode	Off

BOLD

Subtract	Off
StdDev	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On
Distortion Corr.	Off
Contrasts	1
Save original images	On

SIEMENS MAGNETOM Skyra syngo MR D13C

\\USER\AMI-Centre\Toni\SimoS_fruit_Sep2016\GRE Coronal fa20 avg5
 TA:4:18 PAT:Off Voxel size:0.8x0.8x3.0 mm Rel. SNR:1.00 :fl